VALO[®] ORTHO Curing Light


General Information:

VALO[®] ORTHO is an LED curing light for the polymerization of light cured dental materials. With its broadband spectrum, VALO ORTHO is designed to polymerize all light cured products in the wavelength range of 395-480nm. VALO ORTHO has a medical grade, international power supply and is suitable for power outlets from 100 to 240 VAC. The handpiece is designed to rest in a standard dental unit bracket or can be custom mounted using the bracket included with the kit.

Product Components:

- 1 – VALO ORTHO curing light with 7 foot / 2.1 meter cord
- 1 – 9 volt, medical grade, international power supply with 6 foot / 1.8 meter cord and universal plugs
- 1 – Handpiece surface mounting bracket with double stick adhesive tape
- 50 – VALO Barrier Sleeves
- 1 – VALO Light Shield
- 1 – Black Light Lens

Overview of Controls:

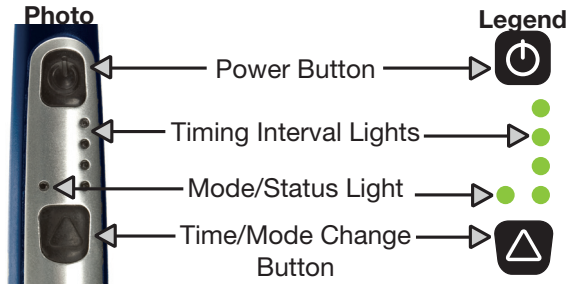Instructions for Use:

- 1 - Connect the 9-volt power cord to the handpiece cord.
- 2 - Plug the power cord into any electrical outlet (100 -240 VAC). The timing lights will illuminate indicating the light is ready for use. ***The VALO ORTHO handpiece beeps when powering on.***
- 3 - Place the VALO ORTHO handpiece into a standard dental unit mounting bracket or accessory mounting bracket until ready to use.
- 4 - Select the desired mode: VALO ORTHO has 3 modes: Standard Power, Xtra Power, and Xtra Power Quadrant. To change modes, hold the Time/Mode Change button for 2 seconds and release. The Mode/Status Light will change to the next mode.

NOTE: VALO ORTHO is programmed to cycle from the Standard Power to the Xtra Power to the Xtra Power Quadrant mode in sequence. For example, to change from the Standard Power mode to the Xtra Power Quadrant mode, it is necessary to cycle into the Xtra Power mode and then to the Xtra Power Quadrant mode.

VALO ORTHO always stores its last completed timing interval in each mode and will default back to that timing interval whenever the modes are changed, or if the unit is unplugged and loses power.

CURING MODE: Standard Power Mode - 1200mW/cm²

WARNINGS AND PRECAUTIONS

DO NOT look directly into the light output. Patient, clinician, and assistants should wear UV orange eye protection when this device is in use.

DO NOT expose soft oral tissues at close proximity. Maintain a safe distance between the lens and the soft tissue.

If a longer cure is needed, allow 10 seconds between consecutive cures or consider a dual-cure product (composite or adhesive).

This unit is programmed for Orthodontic use. If using VALO ORTHO for any other dental procedures, DO NOT exceed 10 seconds in the Standard Power mode.

For any other dental procedure: To avoid overheating the pulp, use two 10-second cures instead of a full 20 second cure.

Mode Power Level	Standard Power Mode - 1200mW/cm²
Power Button	
Mode/Timing LEDs	
Time Button	
Time Options	 5 sec. 10 sec. 15 sec. 20 sec.
To Change Time	Press and release Time Button quickly to cycle through time options.
To Change Modes	Press and hold the Time Button for 2 seconds and release. VALO ORTHO will cycle to the next Mode.
Legend	Solid LEDs   Blinking LEDs  

TIMING INTERVALS: 5, 10, 15, 20 seconds

VALO ORTHO defaults to this mode when it is INITIALLY powered on. The green Mode/Status light is on and the green Timing Lights are solidly illuminated.

To change timing intervals, quickly press the Time/Mode Change Button

Press the Power Button to cure. To stop curing prior to completion of a timing interval, press the Power Button again.

CURING MODE: Xtra Power Mode - 3200mW/cm²

WARNINGS AND PRECAUTIONS

When using the Xtra Power mode, DO NOT expose soft tissue for more than 3, three second cycles.

DO NOT look directly into the light output. Patient, clinician, and assistants should wear UV orange eye protection when this device is in use.

DO NOT expose soft oral tissues at close proximity. Maintain a safe distance between the lens and the soft tissue.

Mode Power Level	Xtra Power Mode - 3200mW/cm²
Power Button	
Mode/Timing LEDs	
Time Button	
Time Options	1 sec. 2 sec. 3 sec.
To Change Time	Press and release Time Button quickly to cycle through time options.
To Change Modes	Press and hold the Time Button for 2 seconds and release. VALO ORTHO will cycle to the next Mode.
Legend	Solid LEDs   Blinking LEDs  

TIMING INTERVALS: 1, 2, 3 seconds

To initiate Xtra Power mode:

From Standard Power mode, press and hold the Time/Mode Change Button for 2 seconds. The green Timing Lights will illuminate and flash. The Status Light will illuminate as a flashing orange light, indicating Xtra Power mode.

To change timing intervals quickly, press the Time/Mode Change Button.

Press the Power Button to cure. To stop curing prior to the completion of a timing interval, press the Power Button again.

To return to Standard Power mode, press and hold the Time/Mode Change Button for 2 seconds, release, hold for 2 seconds, and release.

CURING MODE: Xtra Power Quadrant Mode 3200mW/cm²

The Xtra Power Quadrant Mode is designed to allow the clinician to contiguously cure one quadrant or five teeth without needing to remove the VALO ORTHO handpiece from the oral cavity. Each of the five teeth will be exposed for 3 seconds followed by a 2 second rest.

WARNINGS AND PRECAUTIONS

In Xtra Power Quadrant mode, **DO NOT** expose soft tissue for more than 3, three second cycles. The Xtra Power Quadrant mode has a 2-second safety delay to limit heating during consecutive curing. At the end of the delay, beeping indicates the unit is ready for continued use. If a longer cure is needed, allow 10 seconds between consecutive cures or consider a dual-cure product (composite or adhesive).

DO NOT look directly into the light output. Patient, clinician, and assistants should wear UV orange eye protection when this device is in use.

DO NOT expose soft oral tissues at close proximity. Maintain a safe distance between the lens and the soft tissue.

Mode Power Level	Xtra Power Quadrant Mode 3200mW/cm²
Power Button	
Mode/Timing LEDs	
Time Button	
Time Options	 3 sec. (5 consecutive bursts at 3 seconds each)
To Change Time	Press and release Time Button quickly to cycle through time options.
To Change Modes	Press and hold the Time Button for 2 seconds and release. VALO ORTHO will cycle to the next Mode.
Legend	Solid LEDs   Blinking LEDs  

TIMING INTERVAL: 5 automatically timed continuous bursts of 3 seconds each (Note: there is a 2-second safety delay at the end of each curing cycle).

To initiate Xtra Power Quadrant mode:

From the Standard Power Mode, press the Time/Mode Change Button for 2 seconds, release, press button again for 2 seconds, and release. All of the green Timing Lights will illuminate steady and the orange Status Light will illuminate and flash, indicating Xtra Power Quadrant mode.

Press the Power Button to cure. To stop curing prior to completion of a timing interval, press the Power Button again.

To return to the Standard Power mode, press and hold the Time/Mode Change Button for 2 seconds.

SLEEP MODE: VALO ORTHO will go into POWER SAVE mode after 1 hour of inactivity, as indicated by the green mode light blinking slowly. Press any button to wake VALO ORTHO and begin curing. VALO ORTHO will automatically return to the last setting used.

Follow the guidance of the manufacturer of the light curing material you use in your office regarding curing times and intensity of light needed (See Suggested Curing Card)

WARNINGS AND PRECAUTIONS

Read all instructions before operating this unit. The manufacturer accepts no liability for any damage resulting from the improper use of this unit and/or for any purpose other than those covered by these instructions.

USER/PATIENT SAFETY WARNINGS:

VALO ORTHO emits extremely high intensity light very similar to high intensity quartz halogen lights and must only be used as indicated in this manual.

DO NOT look directly into the light output. Patient, clinician, and assistants should wear UV orange eye protection when this device is in use.

DO NOT expose soft oral tissues at close proximity. Maintain a safe distance between the lens and the soft tissue.

This unit is programmed for Orthodontic use.

For any other dental procedure: To avoid overheating the pulp, use two 10-second cures instead of a full 20 second cure.

In Xtra Power Quadrant mode, DO NOT expose soft tissue for more than 3, three-second cycles. The Xtra Power Quadrant mode has a 2-second safety delay to limit heating during consecutive curing. At the end of the delay, beeping indicates unit is ready for continued used. If a longer cure is needed, allow 10 seconds between consecutive cures or consider a dual-cure product (composite or adhesive).

PRODUCT SAFETY WARNINGS:

DO NOT autoclave.

DO NOT immerse in disinfectant, cleaning solutions, or any kind of liquid.

DO NOT immerse in any kind of ultrasonic bath.

PRECAUTION: This unit may be susceptible to strong magnetic or static electric fields, which could disrupt the programming. If you suspect this has occurred, unplug the unit momentarily and then replug it into the outlet.

Accessory Surface Mounting Bracket Instructions

- 1- Bracket should be mounted to a flat, oil-free surface.
- 2- Clean surface with rubbing alcohol.
- 3- Peel backing off of the bracket's adhesive tape.
- 4- Position bracket so the handpiece lifts upward when removed. Press firmly into place.

Maintenance and Cleaning

VALO ORTHO is a sealed unit with a sapphire-like hard surface and a scratch resistant glass lens. After each use, moisten a gauze or soft cloth with an anti-microbial surface disinfectant and wipe the surface and lens. Periodically check the lens for cured dental resins.

CAUTION: Ensure the VALO ORTHO lens asepsis by using VALO Barrier Sleeves. These sleeves have been designed and optimized specifically for use with VALO ORTHO. In the event that dental resin adheres to the VALO ORTHO lens, use a non-diamond dental instrument to carefully remove the resin.

If using a light meter, Opal Orthodontics by Ultradent recommends checking VALO ORTHO in Standard

Power mode. NOTE: The true numeric output will be skewed due to the inaccuracy of common light meters and the custom LED pack VALO ORTHO uses. Light meters differ greatly and are designed for specific light guide tips and lenses. Their main purpose is not to find an absolute value, but to test consistency in repeated measurements.

ACCEPTABLE CLEANERS:

- Cavicide products
- Isopropyl alcohol-based cleaners
- Ethyl alcohol-based cleaners
- Lysol® disinfectant
- Other non-bleach and non-abrasive disinfectants or cleaners

DO NOT USE

- Strong alkali detergent of any type, including hand soaps and dish soaps.
- Bleach-based cleaners (e.g. - Clorox® , Sterilox®)
- Abrasive Cleaners (e.g. - Comet Cleanser®)
- Acetone-based cleaners (e.g. - nail polish remover, Goo-off™)
- Methyl Ethyl Ketone

Troubleshooting Guide

If the solutions suggested below do not rectify the problem, please call Opal Ortho.

Problem

Possible Solutions

Light will not turn on

1. Press the Time/Mode Change Button or Power Button to wake from Power Save Mode.
2. Check that both cords are firmly connected together and to the electrical outlet.
3. Confirm power to the wall outlet.

Light does not stay on for desired time

1. Check Mode and Timing lights for correct time input.
2. Confirm all cord connections are fully seated.
3. Unplug and re-plug power cord into the electrical receptacle.

Light is not curing resins properly

1. Check lens for residual cured resins/composites. (see “Maintenance and Cleaning”)
2. Using proper orange UV eye protection, verify the LED lights are working. Shine light onto white paper or ceiling to verify that the 4 LEDs are working (1 will be really faint)
3. Check power level with light meter. (See “Maintenance and Cleaning”)
4. Check expiration date on curing resin.

Technical Information

VALO ORTHO LED Curing Light

VALO ORTHO Lens	Diameter 9.6 mm
Effective Curing Wavelength	395nm - 480nm
Light Intensity	Standard Power - 1200mW/cm ^{2*} (+/- 10%) Xtra Power - 3200mW/cm ^{2†} (+/- 10%) Xtra Power Quadrant- 3200mW/cm ^{2†} (+/- 10%)
Wand	Weight - 8 ounces/226 grams (with cord) Length - 9.26 inches/23.5 cm Breadth -.74 inches/1.9 cm Width - .79 inches/ 2 cm Cord length - 7 feet/2.1 meters Ratings - CE, IEC 60601
Power Supply	Output - 9VDC at 2A Input - 100VAC to 240VAC Rating - Medical Grade (UL, CE, RoHS, WEEE) Cord length - 6 feet/1.8 meters

****As measured by a Demetron® L.E.D, Radiometer***

†As measured by a spectrum analyzer

Den-Mat® Sapphire Plasma Arc Curing Light – Irradiance measured at 2,600mW/cm² with a spectrum analyzer

Warranty for VALO ORTHO units:

Opal Ortho hereby warrants that this instrument shall, for a period of 3 years**, conform in all material respects to the specifications therefore as set forth in Ultradent's documentation accompanying the product and be free from any defects in materials/or workmanship. This warranty applies solely to the original purchaser and is not transferable. All defective products are to be returned to Ultradent. There are no user service components of the VALO ORTHO system. Tampering with VALO ORTHO will void its warranty. Removal of the serial label from the cable cord will void VALO ORTHO's warranty.

The VALO warranty does not cover customer caused damage. More specifically the warranty does not cover the cord or the lens. For example; if a VALO is dropped and the lens breaks, the customer would be responsible to pay for any necessary repairs. Also if the cord is vacuumed or rolled over and repairs are necessary the customer would be responsible for the damage.

**With sales receipt indicating the date of sale to the dentist.

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